

Support Exactly Where You Need It.



Why Choose the SuperPole + Bar



Easy to install

No drilling required. Pressure-fit floor-to-ceiling pole.



360° rotation

Support that moves with you. The bar turns & locks every 45°.



Use with only one hand

The SuperBar is easy to use with just one hand. Using Pivot-and-Lock technology, simply lift the bar to move it, and lower it to lock it in place.



Support where you need it

The SuperPole installs between the floor-and-ceiling, so it can be placed exactly where you need it in your home.



Support you can count on

Support available in 300lbs or 450lbs weight capacity.



Works with most ceilings

The standard version works for up to 8' ceilings. Add the Uni-Fit Extender for up to 10' ceilings. Add the Angled Ceiling Plate if your ceilings aren't flat.



www.healthcraftproducts.com
2790 Fenton Road
Ottawa, Ontario, Canada
K1T 3T7

HEALTHCRAFT
Embrace Life.

SuperPole +SuperBar



**Sitting
& Standing
Support!**


Easy Installation

The SuperPole is fast and easy to install. The pole is pressure-fit between your floor and ceiling, **no drilling required**.


1 Turn jackscrew



2 Press top plate to ceiling

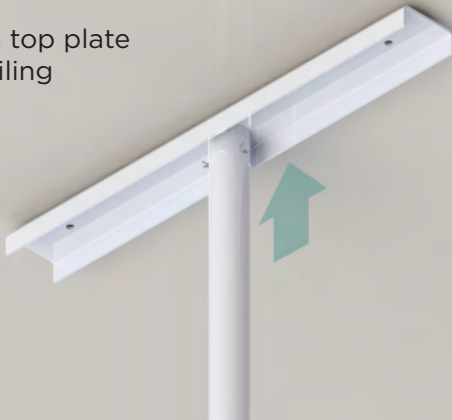

3 Lock with jamnut

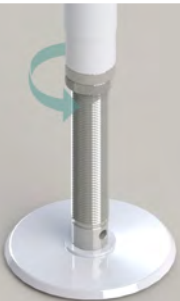

4 Conceal with cover



Moveable Support

The SuperBar offers support all around, turning and locking in place every 45°. Comfortably move the bar with only one hand thanks to its innovative pivot-and-lock design.

